

*Dura-Comfort Table
Contour Assembly Instructions
www.dura-comfort.com*

- 1) Lay table top upside down on the floor.
- 2) Remove all parts from inside the table top.
- 3) Slide legs over bolts installed already in table top. Single leg installs at head of table. The 3-hole legs go at foot of table.
- 4) Install washers and nuts but do not tighten.
- 5) Install armrest bracket to single leg at head and to front set of legs, using bolts, washers and nuts provided. The armrest fits on the inside of legs.
- 6) Place cross-members between legs and install washers and bolts.
- 7) Install padded arm rests to bracket.
- 8) Tighten all nuts. Note: If your table is not level once in place, loosen nuts and put table on floor where it will be then tighten nuts.
- 9) Load paper in holder. Note: If paper doesn't roll freely, flip the roll.
- 10) Enjoy your table!

Dura-Comfort Tables
323 Middle Burnt Fork Rd
Stevensville, MT 59870
866-678-1659

Check List

- 16 – 3” carriage bolts in table
- 8- 2 ½” bolts, 32 washers, 20 nuts
- 4-hole legs w/caps
- 3-hole legs w/caps
- 1 front leg w/cap
- (r-w) 2 cross braces w/inserts
- (r-w) armrest bracket w/caps
- 2 armrests w/bolts
- Manufacturers label
- Face paper hardware – 4 parts (Use only standard 8 ½” roll)
- Max comfort (optional)
- Checked by: _____

