

Dura-Comfort Table

Assembly Instructions for the Massage Table

- () 8 -4 ½" carriage bolts in table
 - () Shelf
 - () 4 Sets of Wood legs with feet
 - () 8 black knobs
 - () 4 Corner brackets mounted & 8 screws
 - () 12 flat washers, 12 lock washers, 12 nuts, 4-2" Phillips flathead bolts, 4-3" Carriage bolts, 4-2 ½" Carriage bolts
 - () Headrest Assembly
 - () Crescent
 - () End Plate
 - () manufacturer label
- Checked by _____

Tools Needed: Phillips screwdriver
7/16" wrench or adjustable wrench

Instructions to install legs:

- 1) Place table top upside down on clean surface.
- 2) Attach leg w/ metal angle over bolts in corner of table top so the angle lines up w/ side of table.
- 3) Install flat washers & nuts. Repeat w/ all 4 legs.
IMPORTANT: Do not tighten the nuts. These must be loose to install shelf.
NOTE: Don't install screws into corner bracket until step 6.

Instructions to install shelf:

- 4) Install 3" carriage bolt through shelf and leg holes furthest from table top. Repeat on all 4 corners so shelf is now in position.
- 5) Install 2" Phillips flat head bolt through counter sunk hole in leg then remaining shelf hole, using flat washer and nut. Repeat on all 4 corners.
- 6) Align legs and shelf w/ table top. Tighten ALL nuts and insert screws through angle bracket into table.
- 7) Install adjustable leg over existing bolt, at desired height black knob. These should remain loose. Install 2 ½" carriage bolt through remaining hole in leg closest to table top w/ black knob. Tighten knobs w/ legs in alignment.
- 8) Install headrest assembly.

If you experience any problems or have any questions regarding installation, feel free to call us!

Phone: 866-678-1659.