

Dura-Comfort Table
Rectangle Assembly Instructions
www.dura-comfort.com

- 1) Lay table top upside down on the floor.
- 2) Remove all parts from inside the table top.
- 3) Slide legs over bolts installed already in table top.
- 4) Install washers and nuts but do not tighten.
- 5) Place cross-members between legs and install washers and bolts.
- 6) Tighten all nuts.
- 7) Mount paper cutter, using screws predrilled into bottom of table. If paper doesn't roll freely, flip roll. Use only standard 8 ½" roll.
- 8) Enjoy your Dura-Comfort Table!

Dura-Comfort Tables
323 Middle Burnt Fork Rd
Stevensville, MT 59870
866-678-1659

Check List

12 – 3" carriage bolts in table
4 - 2 ½" bolts, 16 washers, 12 nuts
4 legs w/caps
(r-w) 2 cross braces w/inserts
Manufacturer's label
Face paper hardware – 4 parts
Max comfort (optional)
Checked by: _____

